

Using your iEar system

Your iEar is a versatile, portable, easy to use assistive listening system that can greatly improve your hearing ability and enhance your quality of life. **Personalise it simply to your activities.** Position it as you want for your hearing pleasure.

- **Listen to a sound source near or far away** by either placing or pointing the transmitter. Our wireless technology will allow you to go as far as 50 metres away with your earpieces still delivering the sound to your ears without compromising on quality, whether or not there is visual contact.
- **Select what you want to hear** from the surround sound or directional microphones on your transmitter or direct audio input from other devices for high fidelity listening.
- **Minimise ambient noise** by amplifying the sound at source using the OFF/Normal/ High sound level switch on the transmitter.


†Users with T switch enabled hearing aids: With the earpieces set in T mode, the iEar acts as a hearing aid add on. It creates your own personal portable induction loop to deliver clear sound devoid of feedback to your ears via your hearing aid.

Your hearing aid will also have to be in T mode. **This setting can be used in any of the following situations as long as you have a T switch enabled hearing aid.**

Here are some suggested settings for a few listening environments. We would encourage you to experiment with the settings to better suit your needs.

Auriplex iEar

Flexible, Versatile, Easy to use, Portable, Life Enhancing

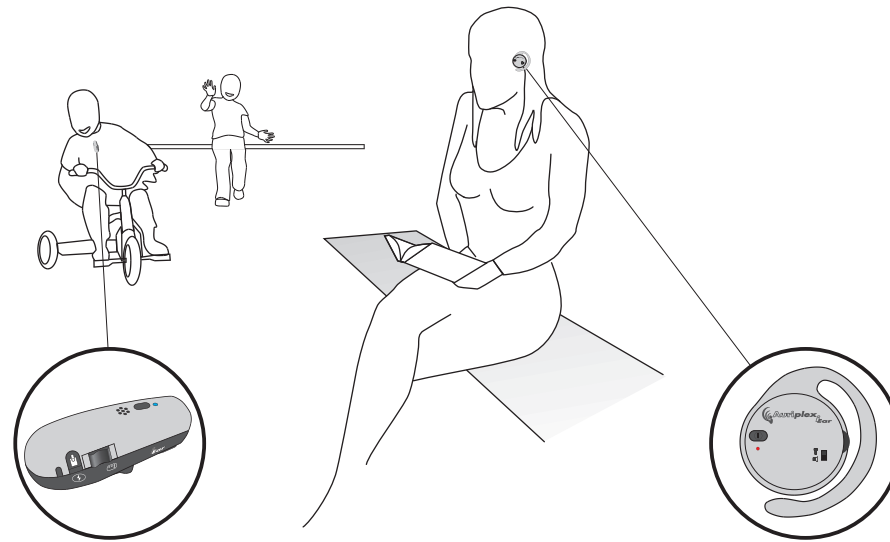

A Quiet setting with Several Appealing sound sources†

E.g: At boardroom business meetings, family gatherings, at the movies.

When you need to hear several people or sound sources in small enough setting with minimal background noise, you could place the transmitter on a table or other appropriate central point from the speakers and interesting sound sources and use the settings below.

Transmitter Settings	
Microphone selector	Red (on the table)
Off/ Normal/ High	Normal

When you need to be aware of your surroundings†

E.g: While commuting and needing to hear ambient sounds e.g. bike riding, walking, or while watching the television show with relatives and friends.

These also apply in case you need to keep track of the surrounding, say while running, or riding your bicycle when you could carry the transmitter on you.

A Noisy place with One Appealing sound source†

E.g: In class, at the restaurant or Café with someone special, at a conference or large meeting, at a party or gathering.



We recommend giving the transmitter to the speaker to wear using the clip or lanyard, and setting the microphone to Surround Sound (Red) and High sound level mode. You could alternatively place the transmitter on the table and direct it towards the speaker in directional (Blue) and High sound level mode.

Transmitter Settings	
Microphone selector	Blue if on a table towards the person Red if on the person
Off/ Normal/ High	High

Direct Audio Input†

E.g: Listening to your personal audio player, the television, your mobile.

Should you want to immerse yourself into a film or melody, you can plug in a 3 mm standard audio cable from your audio or audiovisual equipment into the iEar transmitter direct audio input. This will convey sound at high fidelity to your ears.

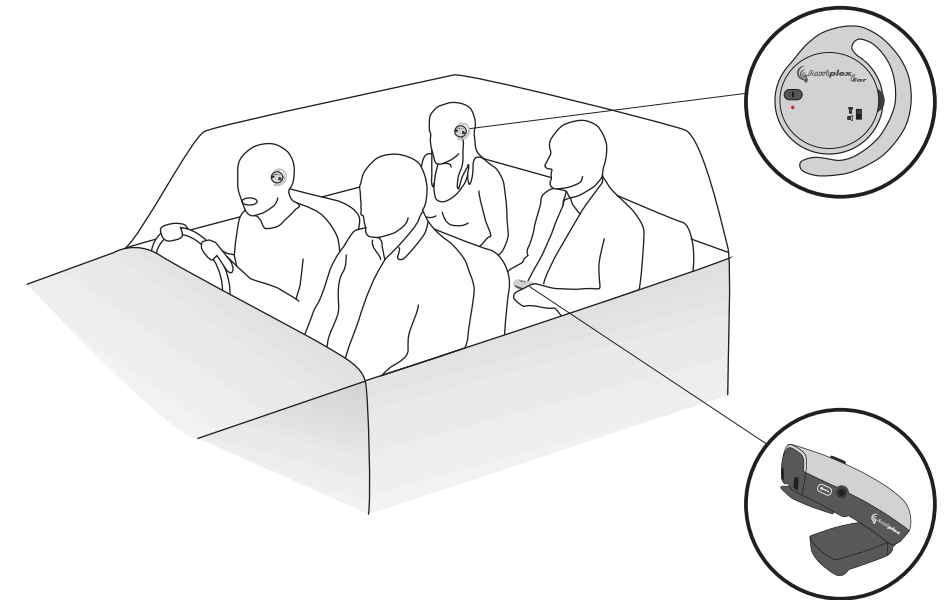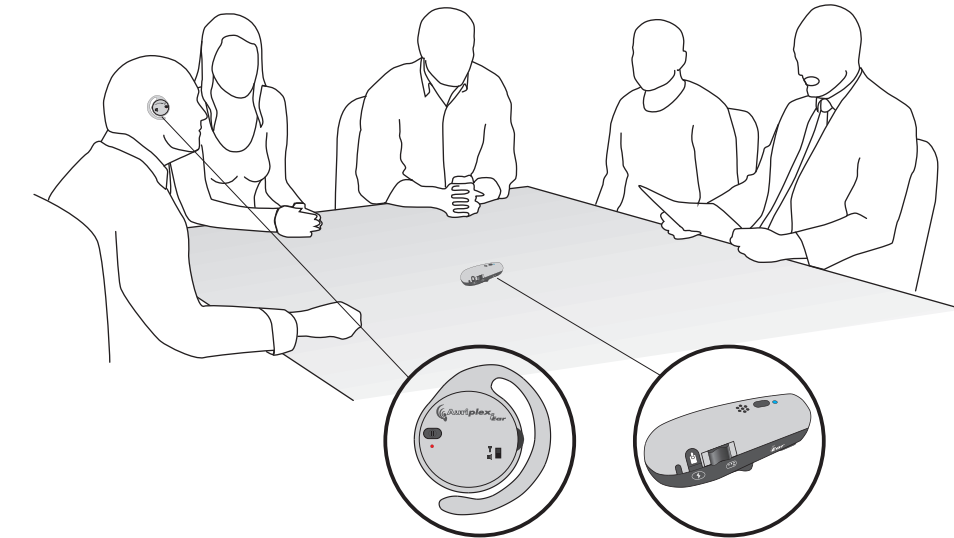


A quiet place with one appealing sound source†

E.g: At a business meeting with one person, at ceremonies.

You could give the transmitter to the person to wear using the clip, lanyard or just in her pocket, and set the microphone to Normal, Omni directional (Red) mode.

Or, you could place it on the table in front of her and direct it towards her in Normal directional (Blue) mode.

Transmitter Settings	
Microphone selector	Blue if on a table, pointing towards the person Red if on the person
Off/ Normal/ High	Normal

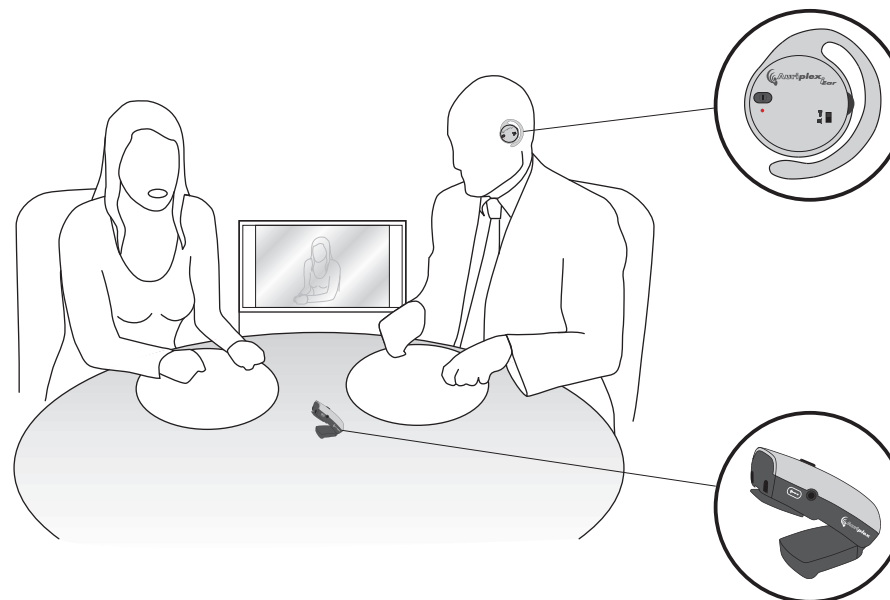

A noisy situation with a small group of people†

E.g: At the restaurant or the pub with a small group of friends, or travelling with friends.

When you need to hear several people or sound sources in a noisy environment, you could place the transmitter as near as possible to the people speaking.

Transmitter Settings	
Microphone selector	Red (on the table)
Off/ Normal/ High	High

Care

Please see your warranty and care card for advice on how to derive a lasting and pleasurable use from your iEar.